911 HIVE

880.06

CE
Regulation (EU)
2017/745

EN

WWW.KONG.IT

Download the digital version

INDEX

| | | |
|--|---|-----------|
|  | 1 - SYMBOLS AND SUPPORT | 4 |
| | 1.1 Symbols | 4 |
| | 1.2 Technical assistance | 4 |
|  | 2 - GENERAL INFORMATION | 5 |
|  | 3 - TECHNICAL FEATURES | 6 |
| | 3.1 Terminology and materials of the parts | 6 |
| | 3.2 Technical data | 7 |
| | 3.3 Optional components and spare parts | 8 |
|  | 4 - SPECIFIC INFORMATION | 10 |
| | 4.1 Intended use | 10 |
| | 4.2 Assembly | 10 |
| | 4.3 Positioning and securing the patient | 12 |
| | 4.4 Transporting the patient | 13 |
|  | 5 - MAINTENANCE AND REPAIR | 15 |
| | 5.1 General Information | 15 |
| | 5.2 Maintenance | 15 |
| | 5.3 Repair | 15 |
|  | 6 - STORAGE | 16 |
|  | 7 - CHECKS, INSPECTIONS AND REVISIONS | 17 |
| | 7.1 Checks | 17 |
| | 7.2 Inspections | 17 |
| | 7.3 Revisions | 17 |
|  | 8 - DURATION AND WARRANTY | 18 |
| | 8.1 Duration | 18 |
| | 8.2 Disposal | 18 |
| | 8.3 Warranty | 18 |
| | 8.4 Law obligations | 18 |
|  | 9 - LABELLING AND SYMBOLS | 19 |
| | 9.1 Medical Device Label | 19 |
| | 9.2 Symbols | 19 |
|  | 10 - DOCUMENTS | 20 |
| | 10.1 Inspections and servicing register | 20 |
| | 10.2 Maintenance and repair register | 21 |
| | 10.3 EU Declaration of conformity | 22 |
|  | 11 - LIST OF APPLIED AND REFERENCE STANDARDS | 23 |
| | 11.1 Applied standards | 23 |
| | 11.2 Standards used as reference | 23 |

CHAPTER

1

SYMBOLS AND SUPPORT

1.1 SYMBOLS

For the sake of comfort and clarity while reading this manual, below are the symbols used for handling important warnings for a proper and safe use of the device.

REQUIREMENT FOR PROPER USE

It identifies the presence of information for proper use of the device.

INFORMATION REQUIREMENT

It identifies the presence of useful and general information which guides the reader towards a conscious use of the device and/or performance of actions.

It identifies that the Medical Device is manufactured, designed and produced in compliance with the provisions of the General Safety and Performance Requirements of Regulation (EU) 2017/745 (Class I medical device, in compliance with classification rule 1 as indicated in the attachment VIII).

1.2 SUPPORT

For information please contact Kong Customer Support Service by:

- telephone +39 0341 630506
- fax +39 0341 641550
- email: safetycare@kong.it

or write to KONG S.p.A. – Via XXV Aprile, 4 – 23804 Monte Marenzo LC - ITALY.

To facilitate support operations, please always communicate or state the serial number (SN) indicated on the label fixed to the Medical Device.

CHAPTER 2

2 GENERAL INFORMATION

The information provided by the manufacturer (hereinafter referred to as information) must be read and clearly understood by the user prior to using the Medical Device (MD hereafter). The information regards the description of the features, performance, assembly, disassembly, maintenance, preservation, disinfection, etc. of the device. Even though the information offers tips on use, this information shall not be considered as a user manual under actual conditions of use.

WARNINGS AND LIMITATIONS OF USE:

- this device shall be strictly used by people who are physically fit, trained (instructed and taught) to use the device and with specific experience regarding moving the patient or, during training activities, by people under direct supervision of the trainers/ supervisors who guarantee the safety thereof,
- do not use the device before fully reading and understanding this user manual,
- strictly follow the manufacturer's information, improper use of the device is hazardous,
- modifying and/or repairing the device is strictly forbidden,
- all checks described in chapter 7 must be carried out prior to and after using the device. In case of any doubt on the efficiency of the device, the user must replace it immediately,
- non-compliant use, deformations, falls, wear, chemical contamination, exposing textile/plastic components/devices to temperature below -30°C or above

- +50°C and metal components/devices to temperatures exceeding 100°C are some examples of causes that can reduce, limit and end the life of the device,
- before use, make sure the load capacity is not greater than that defined for this MD,
- in order to reduce risks of exposure to / transmission of infectious diseases, clean and disinfect the device as indicated in chapter 5,
- improper use of the patient immobilisation systems can jeopardise the safety of the patient,
- always check the compatibility of the devices used alongside the device by consulting the relative manufacturer's information,
- use of spare parts or optional components different from the ones defined for this MD can be hazardous,
- do not expose the device to sources of heat and at contact with chemical substances. Reduce exposure to direct sunlight as much as possible. At low temperatures and in the presence of humidity, formation of ice could reduce flexibility and increase cutting and abrasion-related risks on textile and synthetic devices,
- report to the manufacturer and the competent authority of the member state where the user and / or patient is established any serious incident occurring in relation to the MD.

All our devices are tested/ checked piece by piece in compliance with the procedures laid down by the Quality System certified in accordance with the UNI EN ISO 9001 standard. Laboratory tests, testing, information and standards do not always reproduce the practical result. Thus, the results obtained under the actual conditions of use of the device may differ, even considerably at times. The best information lies in the continuous practical use under the supervision of skilled/expert/qualified people.

Warning: not suitable for use in an ATEX environment (Directive 94/9/CE)

CHAPTER 3

TECHNICAL FEATURES

3.1 TERMINOLOGY AND MATERIALS OF THE PARTS

- A – Aluminium alloy frame, can be disassembled into two parts,
- B – Protective shells made of HDPE,
- C – Polyurethane foam mats,
- D – Polyester webbing for physically securing patient,
- E – Polyester webbing for fixing the mats,
- F – ARIES hanging kit.

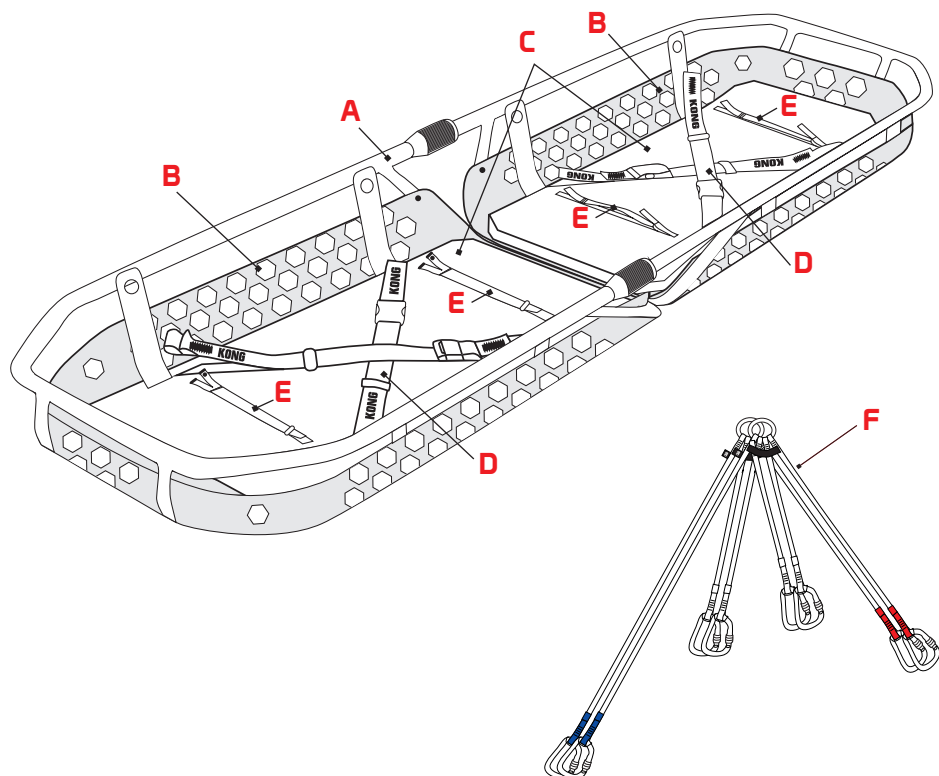
Fig.1

3.2 TECHNICAL DATA

3.2.1 Weight and dimensions

| | |
|-------------------------|--------|
| Length without handles: | 198 cm |
| Width: | 60 cm |
| Weight: | 9,6 kg |

3.2.2 Load capacity

| Mode of use | Load capacity |
|--|---------------|
| Manual transport (fig. 12): | 2400 kg |
| Manual transport with handles 880060KITKK (optional components) - (fig. 2) | 300 kg |
| Lifting and lowering with the hanging kit 880070100KK ARIES (fig. 13) | 2400 kg |
| Transport on slopes (par. 4.4.2) | 300 kg |

3.2.3 Buoyancy force

| Mode of use | Buoyancy force |
|--|----------------|
| River transportation (fig. 4) with kit 880020300KK FLOK-IT | 400 N (~40 kg) |

The safety factor shall be known and defined by the user.

3.3 OPTIONAL COMPONENTS AND SPARE PARTS

The methods of connection and use of the optional components are defined in the relative user instructions.

3.3.1 Optional components

880062KITKK – HANDLE KIT

Handles with an interlocking system that allows the shoulder strap to be connected to the stretcher frame, ensuring resistance and stability during transport.

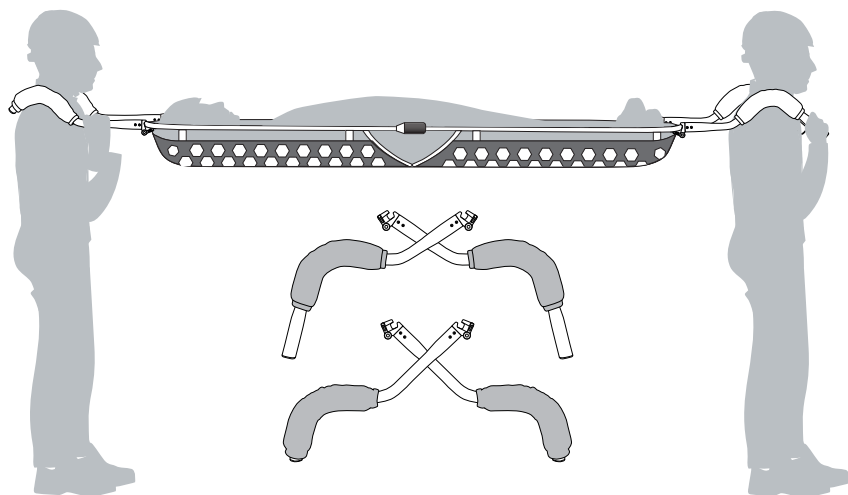
Fig.2

880020300KK FLOK-IT

Floating kit for stretchers, made of three cylinders of different lengths (2 side pieces 1 m long and 1 piece for the head side 40 cm long), each of one equipped with two fastening straps with quick release buckles.

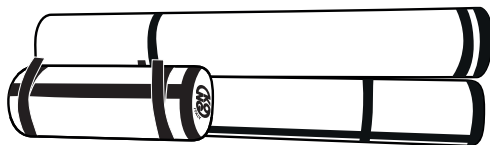
Fig.3

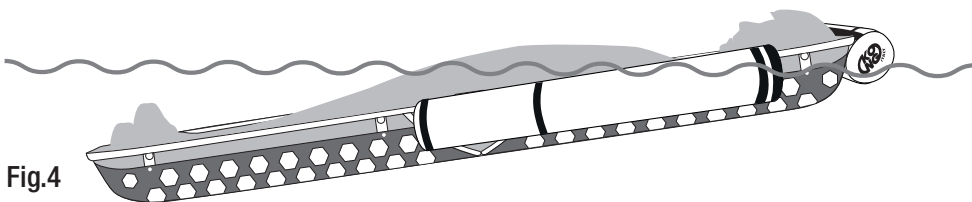
Fig.4

88008000KK WHEEL

Wheel equipped with mechanical disc brakes and a treaded tyre, to be attached to the stretcher in a barycentric position. Facilitates patient transport on flat terrain or on mule tracks.

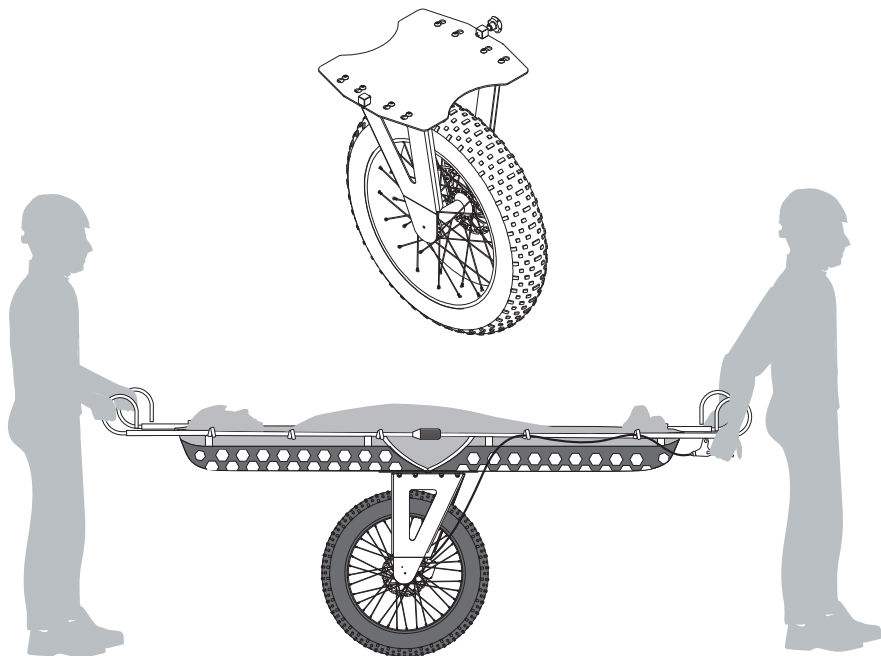
Fig.5

3.3.2 Spare parts

C – Mats.

Fig.6

CHAPTER 4

SPECIFIC INFORMATION

4.1 INTENDED USE

The “911 HIVE” stretcher is a MD particularly suitable for recovering and transporting a patient, which may also be immobilized on the “X-TRIM” spinal boards or the vacuum mattress “VACUUM.” All decisions regarding movement and immobilization of the patient, as well as the duration of these operations, the methods to be adopted and use in conjunction with other devices, must be taken and implemented by expert, trained personnel only. The ARIES hanging kit (F), makes the “911 HIVE” stretcher capable of being lifted by helicopter.

4.2 ASSEMBLY

Assemble the two halves of the structure in the following steps:

- a) bring the two halves closer together (fig. 7),

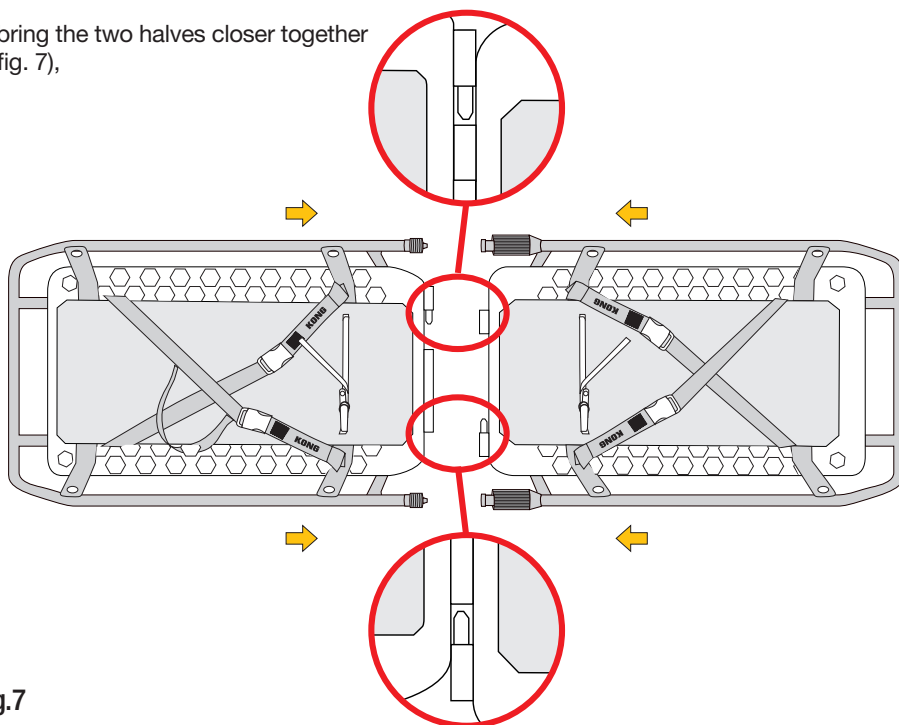
Fig.7

b) lift the two parts centrally and insert the pins into the cylinders (fig. 8),

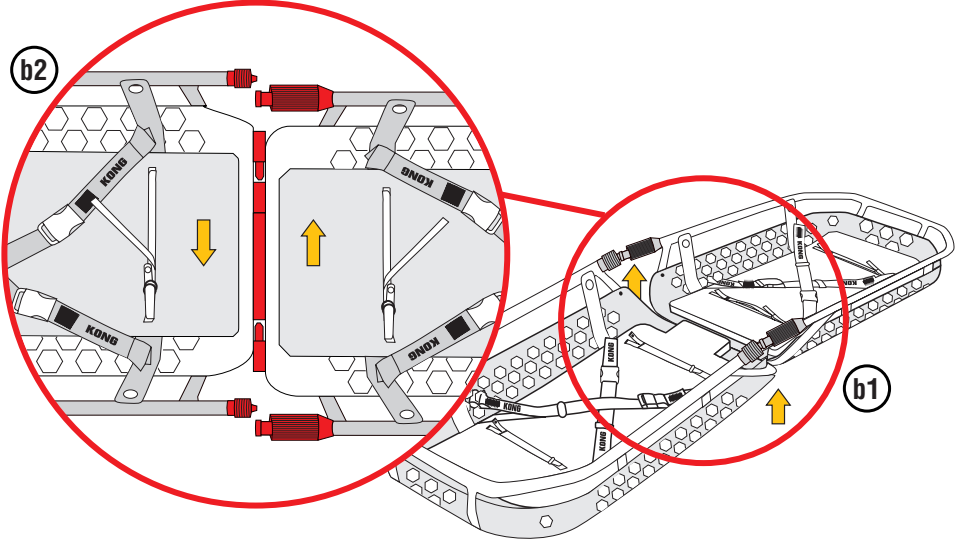
Fig.8

c) fully tighten the two ring nuts (fig. 9).

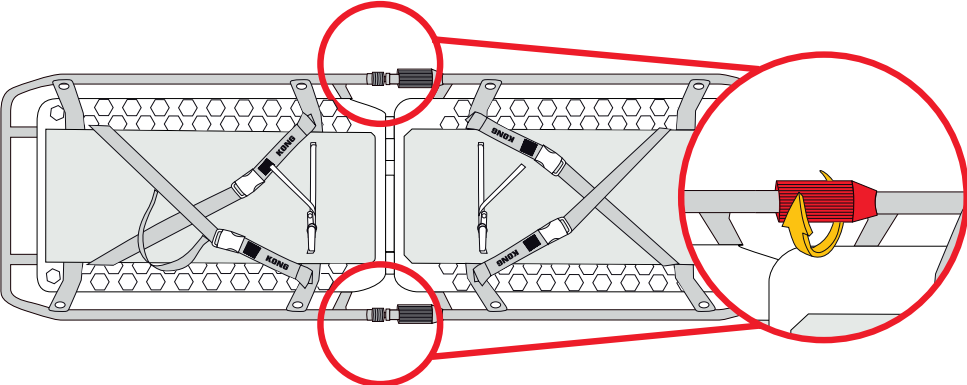
Fig.9

4.3 POSITIONING AND SECURING THE PATIENT

With the stretcher positioned on a flat, stable surface:

- a) unhook the buckles and open the securing straps (fig. 10),
- b) lay the patient in the stretcher, hook on the buckles and tension the straps until they secure the patient (fig. 11).

This procedure must also be used for patients immobilized on the “**X-TRIM**” spinal board or on the “**VACUUM**” vacuum mat.

Avoid prolonged contact of the patient’s skin with the fabrics of the device

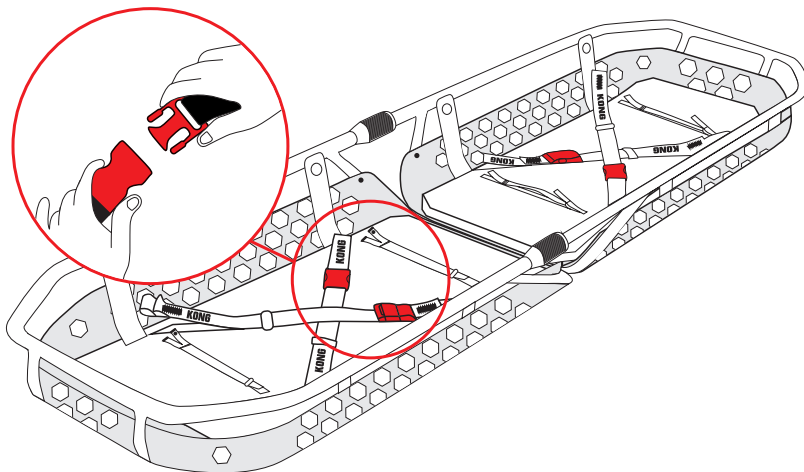
Fig.10

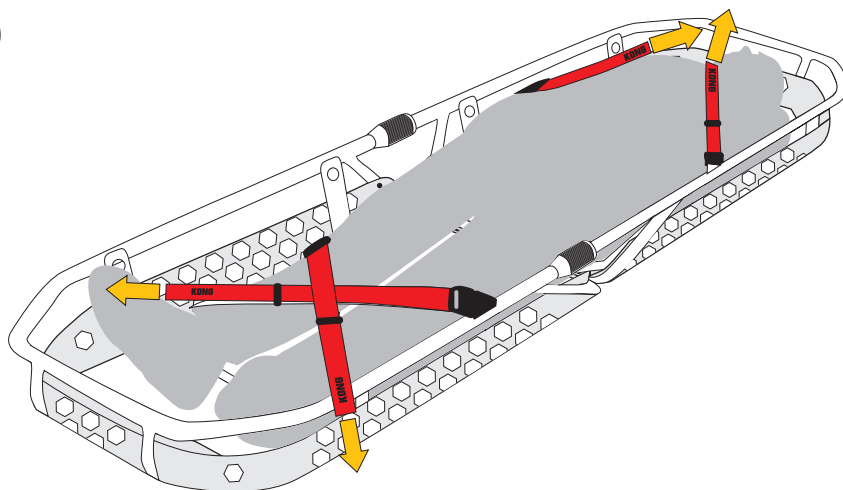
Fig.11

4.4 TRANSPORTING THE PATIENT

The “911 HIVE” stretcher is suitable for manual transport (fig. 12) and for lifting/lowering using the supplied ARIES hanging kit (fig. 13).

The device can be also used in combination with the following optional components:

- a) Handles, for “shoulder” transport (fig. 2),
- b) Flok-it, flotation device (fig. 3 and 4),
- c) Wheel, to facilitate the transport on flat paths and mule tracks (fig. 5).

For transport on slopes, secure the stretcher with a rope, as defined in paragraph 4.4.2.

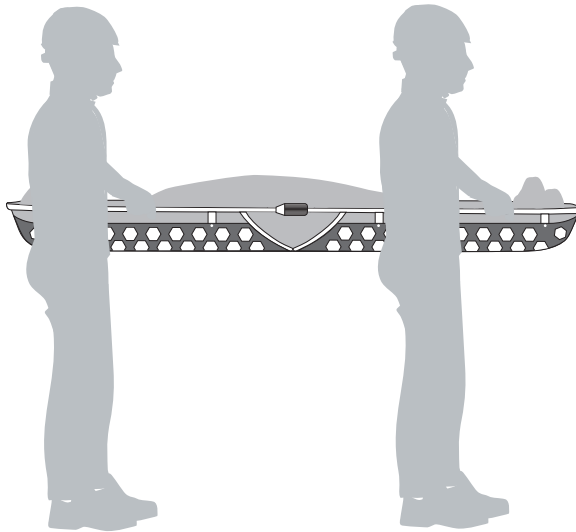
Fig.12

Before any recovery operation, make sure that the weight does not exceed the load capacity defined above!

4.4.1 Connection with the hanging kit

- Connect the hanging kit to the appropriate holes on the side of the stretcher using the eight connectors, while respecting the head-foot coupling indicated on the webbing,
- screw the connector nuts in completely.

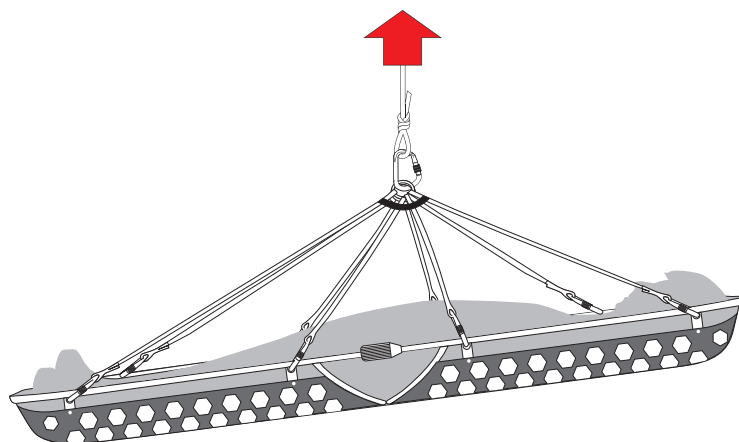
Fig.13

4.4.2 Transport on slopes

For lowering and retrieving on slopes up to 45°, connect the restraining rope to the two highlighted points at the head of the stretcher (fig. 14). It is the responsibility of the operators to define the connection methods, the choice of rope and the relative devices.

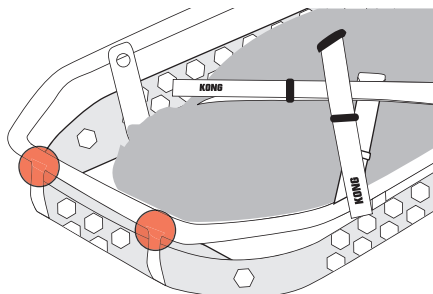
Fig.14

CHAPTER 5

MAINTENANCE AND REPAIR

5.1 GENERAL INFORMATION

This MD is made of material which is highly resistant to wear and external agents. Despite this, the use conditions require maintenance operations and even repair in particular conditions.

Maintenance and repair operations must be recorded in a special register, an example of which is indicated in chapter 10.

5.2 MAINTENANCE

The maintenance operations that must be carried out by the user are cleaning and disinfection (when necessary).

Cleaning: wash the device by hand in lukewarm water (30-32°C) with neutral detergent (pH 5.5-8.5) and allow it to air dry, avoiding direct exposure to the sun and heat sources,

Disinfection: immerse the MD (except for devices in/with parts in Dyneema® and aramid e.g. Kevlar®) for at least 30 minutes in warm water (58-60°C) and allow it to air dry, avoiding direct exposure to the sun and heat sources. Do not bleach, do not wash in a washing machine, do not spin-dry, do not tumble dry, do not iron and avoid sterilizing textile devices in an autoclave.

5.3 REPAIR

Repair shall be strictly carried out by the manufacturer.
The user is allowed to replace the parts mentioned in paragraph 3.3.2 only, with new and genuine parts.

CHAPTER 6

6 STORAGE

After cleaning, disinfecting (if necessary) and drying, store the MD and its accessories in a dry (relative humidity 40-90%), cool (temperature 5-30°C), dark, well-ventilated place, away from sharp edges, chemicals and other harmful conditions.

Do not store this device wet!

CHAPTER 7

CHECKS, INSPECTIONS AND REVISIONS

7.1 CHECKS

Before and after every use, check the device and ensure that:

- it is appropriate for the use it was intended for,
- it has not been subjected to mechanical deformations and it neither has cracks nor shows signs of wear,
- the textile parts do not have cuts, burns, chemical products residues, excessive hair, wear, in particular check areas at contact with the metal components (buckles, rings, etc.);
- the markings, including labels, are legible.

7.2 INSPECTIONS

The device must be inspected at least once a year, starting from the date of first use, by expert personnel trained and approved by KONG S.p.A. The date of the first use and the outcome of the inspection must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

7.3 REVISIONS

The device must be serviced by KONG S.p.A., or by expert personnel trained and approved by KONG S.p.A. if:

- any malfunctions are detected,
- the result of the pre and post use check or of the inspections is negative.

The outcome of the servicing must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

The revised device is guaranteed for one year from the revision date.

CHAPTER 8

DURATION AND WARRANTY

8.1 DURATION

Carefully read the “Warnings and limitations of use” paragraph in chapter 2. The duration of this device depends on the outcome of the at least one-yearly inspection (paragraph 7.2) and possible servicing. The hanging kit, even if not used, must be replaced after 10 years from the first use and in any case no later than 12 years from the date of manufacture.

Important: any devices that have not passed the pre-use, post-use and periodic inspection must be eliminated and made unusable.

8.2 DISPOSAL

Follow the rules in force in the country of use and the waste disposal procedures of the reference hospital facility for proper disposal.

8.3 WARRANTY

The manufacturer guarantees the compliance of the device with the regulations in force at the time of production. Warranty covering defects is limited to defects regarding raw materials and manufacturing defects. The warranty does not cover normal wear, oxidation, damage caused by non-compliant use and/or in competition, improper maintenance, transportation, preservation or storage. Warranty will be immediately deemed null and void should the device be modified or tampered with. Validity corresponds to the legal validity of the country where the device was sold, as from the date of sale, by the manufacturer. No claims shall be raised against the manufacturer once the aforementioned period has expired. Any repair or replacement request under warranty must be accompanied by a proof of purchase. Should the defect be recognised, the manufacturer undertakes to repair or, replace or refund the device at his discretion. The manufacturer shall not be held liable beyond the invoice price of the device in any case whatsoever.


8.4 LAW OBLIGATIONS

Professional and free time activities are often regulated by special national laws which can impose limits and/or obligations concerning the use of these devices. The user has the obligation to know and apply such laws which could provide for limits different from those indicated in this information.

CHAPTER 9

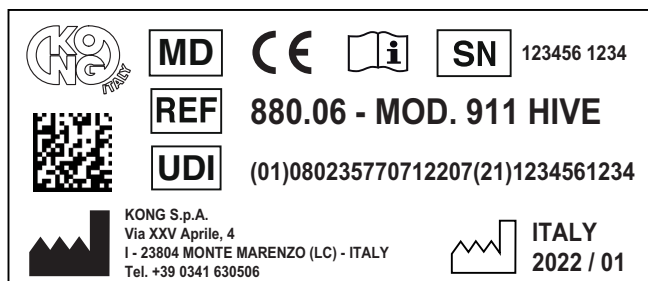LABELLING AND SYMBOLS

9.1 MEDICAL DEVICE LABEL

Any changes to the positioning of the symbols does not change the content thereof.

9.2 SYMBOLS

Identification and traceability

| | |
|---|--|
|  | Medical Device |
|  | Product identification code |
|  | Unique Device Identifier |
|  | Manufacturer's identification |
|  | Unique serial number |
|  | Country, year and month of production. |

Data matrix UDI
 Readable with the Apps:
 iGepir - BarValid - Barcode Scanner

Information and safety

| | |
|---|---|
|  | Consult the user manual |
|  | Compliant with Regulation (EU) 2017/745 |

CHAPTER 10

DOCUMENTS

10.1 INSPECTIONS AND SERVICING REGISTER

| STRETCHER - Class I MD | | | | |
|----------------------------------|----------------|--------------------|----------------|-------------------|
| REF. | 880.06 | MOD. | 911 HIVE | |
| UDI-DI | 08023577071220 | SN | | |
| Date of first use | | | | |
| INSPECTIONS AND SERVICING | | | | |
| DATE | I/S | Description | Outcome | Supervisor |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |

10.2 MAINTENANCE AND REPAIR REGISTER

| STRETCHER - Class I MD | | | | |
|-----------------------------------|----------------|--------------------|----------------|-------------------|
| REF. | 880.06 | MOD. | 911 HIVE | |
| UDI-DI | 08023577071220 | SN | | |
| Date of first use | | | | |
| MANUTENZIONI E RIPARAZIONI | | | | |
| DATA | M/R | Description | Outcome | Supervisor |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |
| | | | | |

10.3 EU DECLARATION OF CONFORMITY

The EU Declaration of Conformity is available for download at www.kong.it

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CHAPTER 11

11.1 APPLIED STANDARDS

- EN 597-1: 2015
Furniture - Assessment of the ignitability of mattresses and upholstered bed bases - Part 1: Ignition source smouldering cigarette
- EN 1865-1: 2010 + A1: 2015
Patient handling equipment used in road ambulances - Part 1: General stretcher systems and patient handling equipment
- EN 62366-1: 2015 + A1: 2020
Medical devices - Application of usability engineering to medical devices
- EN ISO 846:2019
Plastics - Evaluation of the action of microorganisms
- EN ISO 10993-1: 2020
Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process
- EN ISO 10993-5: 2009
Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity
- EN ISO 10993-23: 2021
Biological evaluation of medical devices - Part 23: Tests for irritation
- EN ISO 14971: 2019
Medical devices - Application of risk management to medical devices
- EN ISO 15223-1: 2021
Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements
- EN ISO 20417: 2021
Medical devices - Information to be supplied by the manufacturer.

11.2 STANDARDS USED AS REFERENCE

- EN ISO 13485: 2016
Medical devices - Quality management systems - Requirements for regulatory purposes
- EN 13718-1: 2014 + A1: 2020
Medical vehicles and their equipment - Air ambulances - Part 1: Requirements for medical devices used in air ambulances.

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